
English	3
Bosanski	4
Čeština	5
Dansk	6
Deutsch	7
Eestlane	8
Español	9
Français	10
Gaeilge	11
Hrvatski	12
Italiano	13
Latvietis	14
Lietuvis	15
Magyar	16
Nederlands	17
Polskie	18
Português	19
Română	20
Slovenščina	21
Slovenský	22
Suomi	23
Svenska	24
Türk	25
Ελληνικά	26
български	27
русский	28
Український	29
한국어	30
日本語	31
简体中文	32

DL5 Plus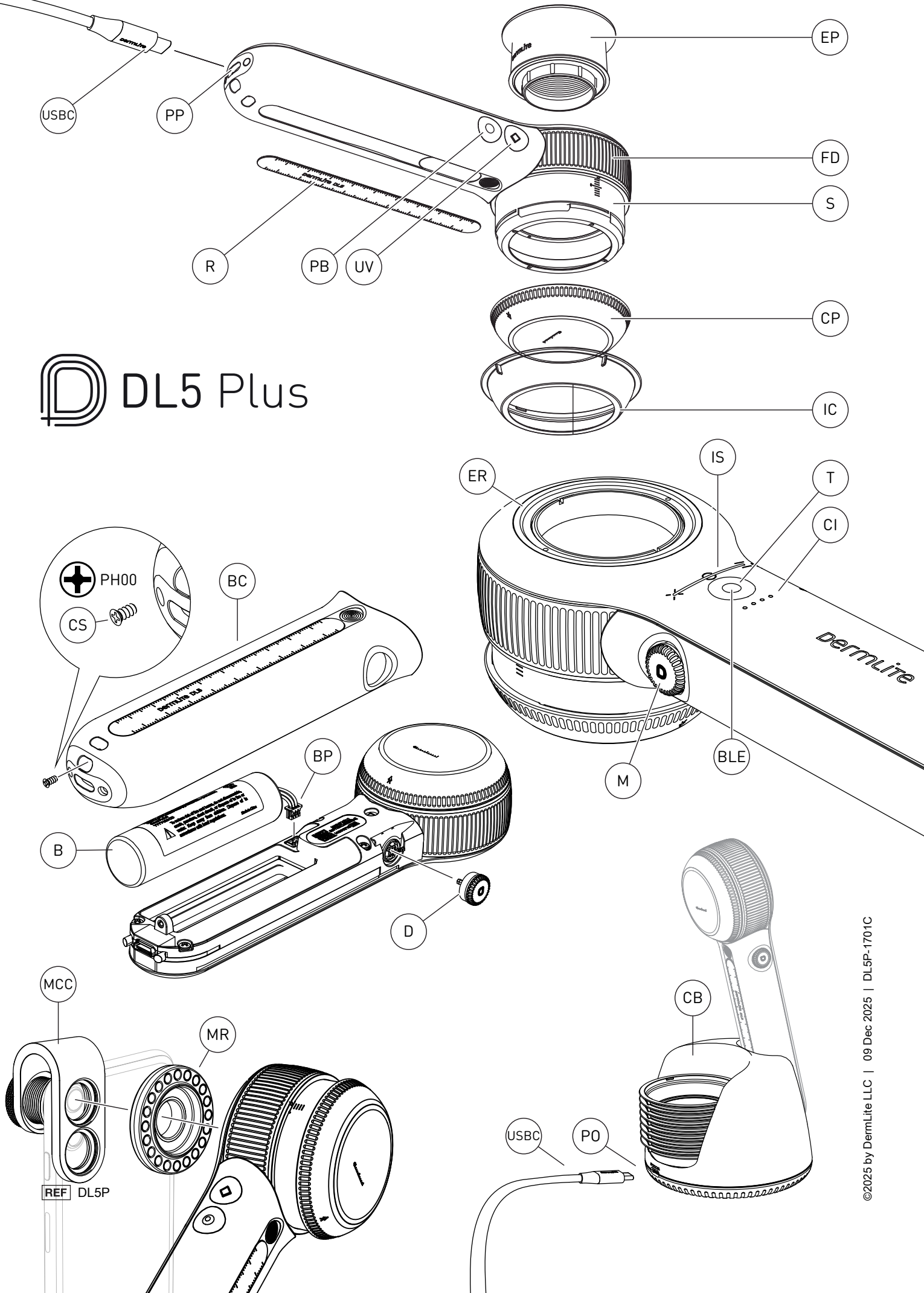

Instructions

Intended Use

This device is intended for medical purposes to illuminate body surfaces. It is used for non-invasive visual examination of intact skin and assists clinicians in the assessment and differentiation of skin abnormalities by enhancing the visualization of surface and subsurface structures.

This battery-operated product is designed for external examination only within professional healthcare facilities by medical professionals.

Check the correct operation of the device before use!

Do not use the device if there are visible signs of damage, malfunction, or the device operates in a manner not consistent with what is described in these instructions for use.

CAUTION: Do not look directly into the LED light. Patients must close their eyes during examinations.

In case of a serious incident with the use of this device, notify DermLite immediately and, if required by local regulations, your national health authority.

CAUTION: Do not use the device in fire or explosive risk area (e.g., oxygen-rich environment).

CAUTION: UV Light. Do not look directly at light.

CAUTION: This product contains magnets. Do not use in proximity of pacemakers or other magnetically sensitive devices or items.

WARNING: This product can expose you to chemicals including methylene chloride and hexavalent chromium, which are known to the State of California to cause cancer or reproductive toxicity. For more information go to www.P65Warnings.ca.gov.

Electromagnetic Compatibility

This device complies with the EMC Emissions and Immunity level requirements of the standard IEC 60601-1-2:2014. The emission characteristics of this equipment make it suitable for use in professional healthcare environment as well as residential environment (CISPR 11 Class B). This equipment offers adequate protection to radio communication service. In the rare event of interference to the radio communication service, the user might need to take mitigation measures, such as relocating or re-orienting equipment.

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING: Use of accessories other than those provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING: Portable RF transmitters should be used no closer than 30 cm (12 inches) to any part of the device. Otherwise, degradation of the performance of this equipment could result.

Do not use this device during magnetic resonance imaging (MRI) or in an MRI environment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Any changes/modifications to this equipment not approved by DermLite could void the user's authority to operate the equipment.

IMPORTANT: Before initial use, please charge the internal lithium ion battery. See "Battery and Charging" for details.

The DermLite DL5 dermatoscope is a smartphone-compatible pocket dermatoscope with 10x magnification for visualizing skin structures using variable polarized light and UV illumination.

Hold your DermLite DL5, with the LEDs aimed in the direction of the lesion to be examined, approximately 12 mm above the skin. Push the main button (M) to turn on the device in cross-polarized mode. Look through the lens and move the device closer or farther away from the skin to focus the image. For dermoscopy with skin contact, rotate the Focus Dial (FD) to extend the Spacer (S) and focus the image. Push and hold the main button (M) for about 1 second to turn off the device. To save power, the unit automatically turns off after 3 minutes.

Variable Polarization

Tap the main button (M) to toggle between cross-polarized and non-polarized illumination. Turn the main button to move through the range of 9 polarization settings, from cross-polarized via non-polarized to parallel-polarized light. The indicator strip (IS) reflects the current polarization status, whereas "X" marks cross-polarized, "0" represents non-polarized, and "=" stands for parallel polarization.

PigmentBoost® Illumination

While white light is active, tap (PB) to activate the orange-colored PigmentBoost lighting in addition to white light. The indicator LED next to the PB button (PB) will illuminate. Tap again to deactivate. To adjust the ratio of orange to white lighting push and hold the PB button (PB) and turn the main button (M) while PigmentBoost is activated.

PigmentBoost Plus

To visualize pigmented structures, push and hold (PB) for 1 second while the device is on to activate the orange-colored LEDs exclusively. Turn the main button (M) to adjust orange-colored LED brightness. Tap (PB) again to return to the previously selected white-light mode.

Torch

To illuminate larger skin areas or body cavities, tap the torch button (T) to activate an ultra-bright white non-polarized LED. The BLE indicator will illuminate white. Turn the main button (M) to adjust torch brightness.

Dermoscopic Brightness Adjustment

To adjust the brightness of the white dermoscopic LEDs, hold the torch button (T) or UV button (UV) and simultaneously turn the main button (M). The indicator strip (IS) reflects the current brightness level during the adjustment.

Eyepiece

For optimal viewing affix the included eyepiece (EP) at the extension ring (ER).

Smartphone Dermoscopy

For dermoscopy using a smartphone, fasten the MagnetiConnect adapter (MCC) to your phone, ensuring concentric alignment with the desired camera. Attach your DL5 to the MCC ring. Prior to taking images, extend the spacer (S), and wipe the glass contact plate (CP) clean.

This dermatoscope may be controlled wirelessly from a compatible smartphone app.* To pair it with your smartphone, push and hold the torch button (T) for 3 seconds while off and follow your app's instructions. The BLE indicator (BLE) flashes blue while in pairing mode and lights up solid when actively connected.

Wherever wireless connectivity must be disabled, hold the (UV) and (PB) buttons simultaneously for 3 seconds to disable this functionality. The BLE indicator (BLE) flashes 6 times to confirm. To re-enable, enter pairing mode by holding the torch button for 3 seconds until the BLE indicator flashes.

Upon turning off the DL5 lights, the BLE indicator flashes once to indicate that wireless connectivity is enabled and it is paired to a smartphone.

*Subject to availability

Reset

To reset the device to its factory defaults and remove all paired devices from memory press the following button sequence while the device is off: (UV)-(PB)-(PB)-(UV)-(PB)-(PB)-(UV). The BLE indicator (BLE) will flash to confirm.

Vacation Mode

This DL5 is designed to be energy-efficient but still uses a small amount of power when idle. To minimize power usage during extended non-use, activate Vacation Mode by pressing and holding the main button (M) for 3 seconds. The indicator strip (IS) will illuminate then fade, confirming Vacation Mode is activated. To exit Vacation Mode, simply press the main button.

Removing the Contact Plate

Turn the glass contact plate (CP) counterclockwise to detach, turn it clockwise to reattach.

Ruler

The ruler (R) is magnetically attached to the handle. To remove it, press down in the center of the ruler.

IceCap® Disposable Caps

To support your infection control measures, snap one of the included IceCap (IC) disposable covers over the installed faceplate (CP). To remove and dispose, simply pull it off.

Battery and Charging

CAUTION: This device uses a special 3.7V 3000mAh lithium ion battery, which can only be purchased from DermLite or from an authorized DermLite dealer. Do not under any circumstances use another battery other than the one designed for this unit.

After hundreds of uses the battery will deteriorate, which may result in reduced capacity and/or lithium-ion cell expansion. Monitor for signs of a deteriorated battery which may include reduced battery life, unexpected shutdowns, erratic behavior, or sudden power loss. If observed, stop use of the device and do not charge the battery. Replace the battery and dispose of the original in accordance with local regulations.

This device has a four-level Charge Indicator (CI). When turned on and fully charged, four LEDs light up. With less than 75% battery, three LEDs illuminate; with less than 50%, two LEDs; and with less than 25%, one LED. To charge, connect the supplied USB cable to the Charging Port (PP) and a 5V IEC 62368-1 compliant USB port. During charging, the CI pulses, with the number of pulsing LEDs indicating charge level. When charging is complete, all four LEDs light up.

To change the battery (B), ensure the USB cable is disconnected from the charging port (PP), pull off the dial (D) by grabbing it at the fingernail notch, unscrew the battery cover screw (CS) using the tool supplied with each DL5 replacement battery, and remove the battery cover (BC) by sliding it back. Disconnect the small white battery plug (BP) and remove the battery (B). Insert a new battery, reconnect the battery plug, reattach the battery cover (BC), secure it with the cover screw (CS), and push the dial (D) in until it snaps into place.

Charging Base

To charge your DL5 dermatoscope using the charging base, attach one end of the supplied USB cable to the power port (PO) on the back of the charging base (CB), and the other end to any IEC 60950-1 or IEC62368-1 compliant (5V) USB charger. Place your dermatoscope into the charging base, with the contact plate (CP) facing towards the charging base. The charging base may also be used for IceCap storage.

Troubleshooting

Please check www.derm-lite.com for the most current troubleshooting information. If your device requires servicing, visit www.derm-lite.com/service or contact your local DermLite dealer.

Care and Maintenance

CAUTION: No modification of this equipment is allowed.

Your device is designed for trouble-free operation. Repairs shall be made only by qualified service personnel. It is recommended to check for correct operation of the device every six months.

Cleaning

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.) prior to use on a patient. The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

Warranty: 5 years for parts and labor. The battery is warranted for 1 year.

Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

This set includes:

DermLite DL5 Plus (16 white LEDs, 4 UV LEDs, 4 PigmentBoost LEDs, 1 white Torch LED, 32 mm lens system with 10x magnification, variable polarization system, wireless connectivity, retractable spacer, removable faceplate with 10 mm reticle, steel ruler), leather belt case, USB charging cable, charging base, eyepiece, MCC mobile device adapter, plastic belt holster (H), 5 IceCaps, microfiber cloth.

Technical Description

Visit www.derm-lite.com/technical or contact your local DermLite dealer.

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DermLite LLC
E. C. REP Ltd
Responsible Person Limited
E C Rep GmbH Max-