

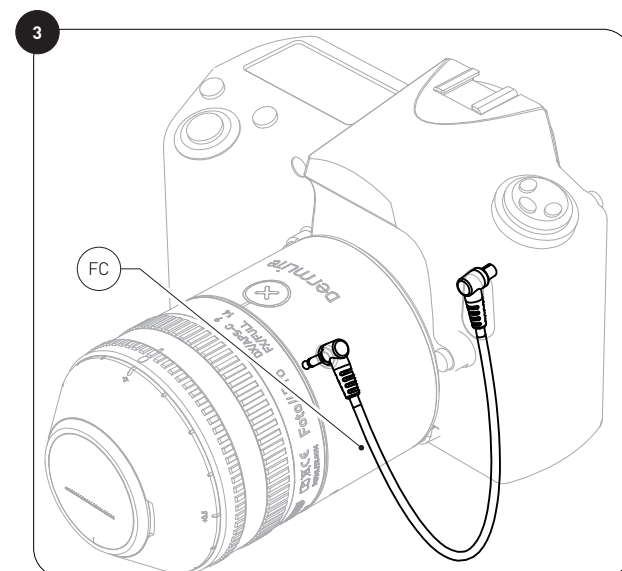
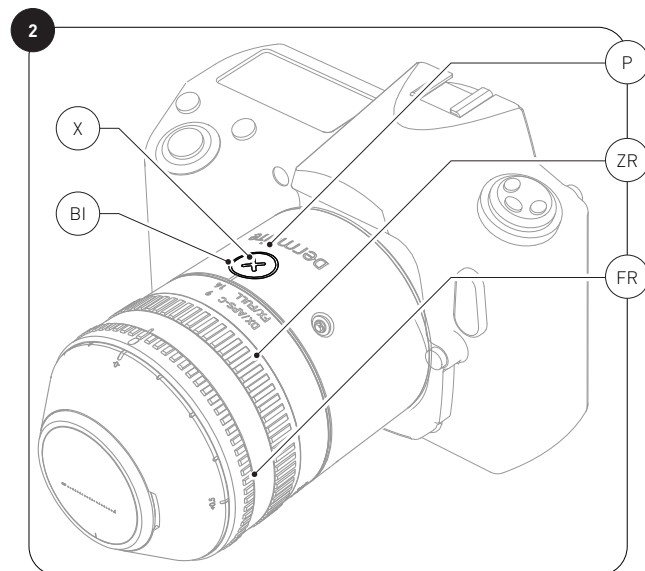
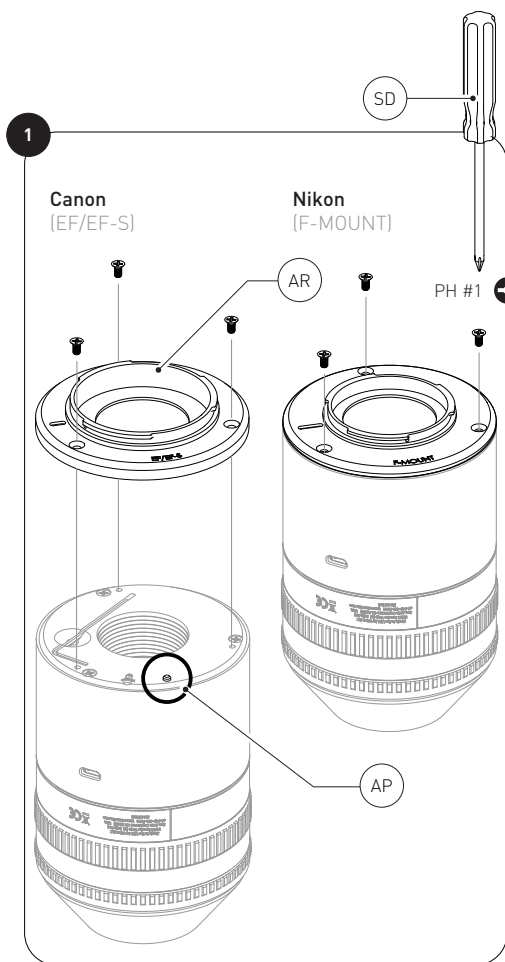
dermLite[®] Foto//Pro

English	3
Bosanski	4
Čeština	5
Dansk	6
Deutsch	7
Eestlane	8
Español	9
Français	10
Gaeilge	11
Hrvatski	12
Italiano	13
Latvietis	14
Lietuvis	15
Magyar	16
Nederlands	17
Polskie	18
Português	19
Română	20
Slovenščina	21
Slovenský	22
Suomi	23
Svenska	24
Türk	25
Ελληνικά	26
български	27
русский	28
Український	29
한국어	30
日本語	31
简体中文	32

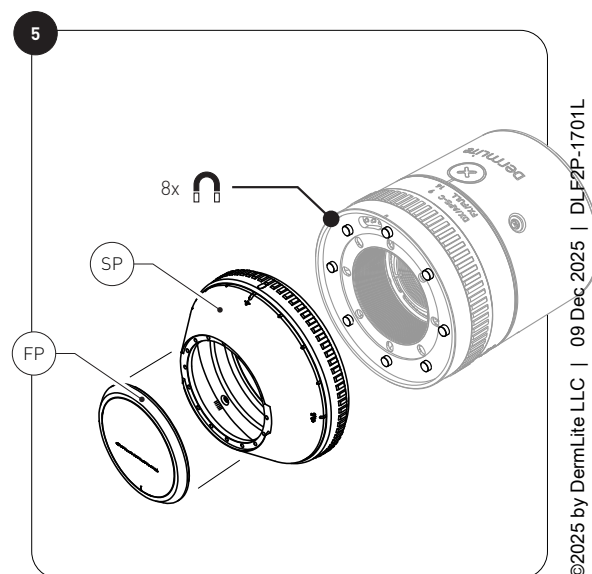DermLite®
Foto//Pro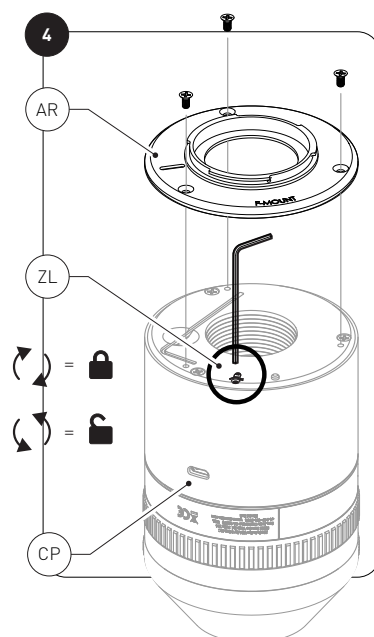

REF DLF2P

Instructions

Intended Use

This device is intended for medical purposes to illuminate body surfaces. It is used for non-invasive visual examination of intact skin and assists clinicians in the assessment and differentiation of skin abnormalities by enhancing the visualization of surface and subsurface structures.

This battery-operated product is designed for external examination only within professional healthcare facilities by medical professionals. Check the correct operation of the device before use!

Do not use the device if there are visible signs of damage, malfunction, or the device operates in a manner not consistent with what is described in these instructions for use.

CAUTION: Do not look directly into the LED light. Patients must close their eyes during examinations.

In case of a serious incident with the use of this device, notify DermLite immediately and, if required by local regulations, your national health authority.

CAUTION: Do not use the device in fire or explosive risk area (e.g., oxygen-rich environment).

CAUTION: This product contains magnets. Do not use in proximity of pacemakers or other magnetically sensitive devices or items. **WARNING:** This product can expose you to chemicals including methylene chloride and hexavalent chromium, which are known to the State of California to cause cancer or reproductive toxicity. For more information go to www.P65Warnings.ca.gov.

Electromagnetic Compatibility

This device complies with the EMC Emissions and Immunity level requirements of the standard IEC 60601-1-2:2014. The emission characteristics of this equipment make it suitable for use in professional healthcare environment as well as residential environment (CISPR 11 Class B). This equipment offers adequate protection to radio communication service. In the rare event of interference to the radio communication service, the user might need to take mitigation measures, such as relocating or re-orienting equipment.

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING: Use of accessories other than those provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING: Portable RF transmitters should be used no closer than 30 cm (12 inches) to any part of the device. Otherwise, degradation of the performance of this equipment could result.

IMPORTANT: Before initial use, please charge the internal lithium ion battery. See "Battery and Charging" for details.

DermLite Foto II Pro is a dermoscopy lens designed to let you capture high-resolution dermoscopic images with any Canon or Nikon SLR.

Directions for Use (Fig. 1-5)

Fig 1. The device is ready to be used with Nikon SLR cameras. For use with Canon cameras, detach the installed F-MOUNT adapter and replace it with the EF-EF-S adapter, making sure the alignment pin (AP) is seated in its mating hole.

Carefully place the device with the faceplate (FP) onto the skin. Fig. 2: To activate the unit in polarized mode, push on the X button. To activate non-polarized illumination, push it again. You may change the field-of-view by rotating the zoom ring (ZR) to either the fully zoomed-in or zoomed-out position; avoid a midrange zoom setting as this may result in out-of-focus images. The numbers on the zoom ring indicate image frame height in mm for both full-frame sensors as well as cameras with a smaller sensor size.

The device captures images with a very shallow depth of field which enables precise focusing on the area of interest. The focus ring's (FR) "0" mark has been factory-calibrated to result in sharp images in most cases.

To make fine focus adjustments, activate the "Live View" mode on your camera, if available, then digitally zoom in using your camera controls while turning the focus ring (FR) one marking to the right or left.

Fig. 3: By attaching the flash sync cable (FC) as shown, the device will automatically change polarization mode after each photo.

Fig. 4. Zoom lock (ZL): In order to achieve repeatability, you may lock the zoom ring using the included hex key. Simply remove the adapter ring (AR), turn the key as shown, then reattach the ring.

Fig. 5 The spacer (SP) and faceplate (FP) are attached magnetically and may be easily removed for cleaning. To turn off the unit, push and hold the button for about 1 second.

Battery and Charging

CAUTION: This device uses a special 3.7V 1480 mAh (2pcs) lithium ion battery, which can only be purchased from DermLite or from an authorized DermLite dealer. Do not under any circumstances use another battery other than the one designed for this unit.

After hundreds of uses the battery will deteriorate, which may result in reduced capacity and/or lithium-ion cell expansion. Monitor for signs of a deteriorated battery which may include reduced battery life, unexpected shutdowns, erratic behavior, or sudden power loss. If observed, stop use of the device and do not charge the battery. Replace the battery and dispose of the original in accordance with local regulations.

This device is equipped with a battery indicator (CI). When the unit is turned on and fully charged, the indicator lights up green; when less than 20% of battery life remains, the indicator is orange while a red glow shows that less than 10% of battery life remains. To charge your device, connect the supplied Micro-USB to USB cable to the Charging Port

(CP) and any IEC 60950-1 compliant (5V) USB port. The battery indicator glows red to indicate charging. When charging is complete, it illuminates green. From a full charge, the device will allow approximately 2-4 hours of operation.

After years of use, you may wish to replace the battery, which is only available from DermLite or an authorized DermLite dealer. To change the battery, contact DermLite or your local DermLite dealer.

Troubleshooting

Please check www.derm-lite.com for the most current troubleshooting information. If your device requires servicing, visit www.derm-lite.com/service or contact your local DermLite dealer.

Care and Maintenance

CAUTION: No modification of this equipment is allowed.

Your device is designed for trouble-free operation. Repairs shall be made only by qualified service personnel. It is recommended to check for correct operation of the device every six months.

Cleaning

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.) prior to use on a patient. The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

DermLite Foto II Pro includes:

Foto II Pro unit with removable spacer and faceplate, USB cable, Nikon and Canon adapter rings, flash sync cable, international charger.

Warranty: 2 years for parts and labor. The battery is warranted for 1 year.

Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

Technical Description

Visit www.derm-lite.com/technical or contact your local DermLite dealer.

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DermLite LLC
E. C. REP Ltd
UK CA Responsible Person Limited
CH REP E C Rep GmbH Max-