

English	3
Bosanski	4
Čeština	5
Dansk	6
Deutsch	7
Eestlane	8
Español	9
Français	10
Gaeilge	11
Hrvatski	12
Italiano	13
Latvietis	14
Lietuvis	15
Magyar	16
Nederlands	17
Polskie	18
Português	19
Română	20
Slovenščina	21
Slovenský	22
Suomi	23
Svenska	24
Türk	25
Ελληνικά	26
български	27
русский	28
Український	29
한국어	30
日本語	31
简体中文	32

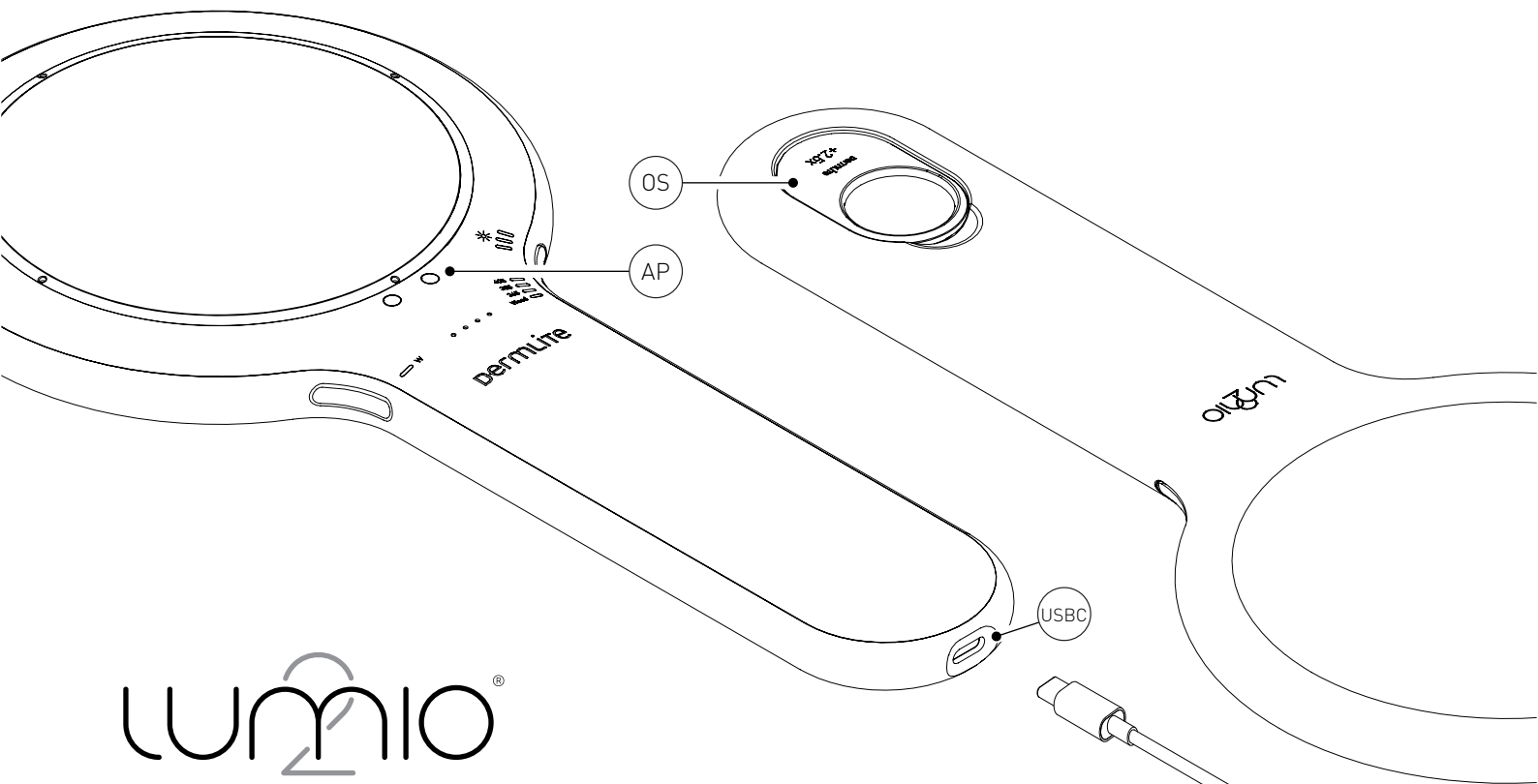lumio®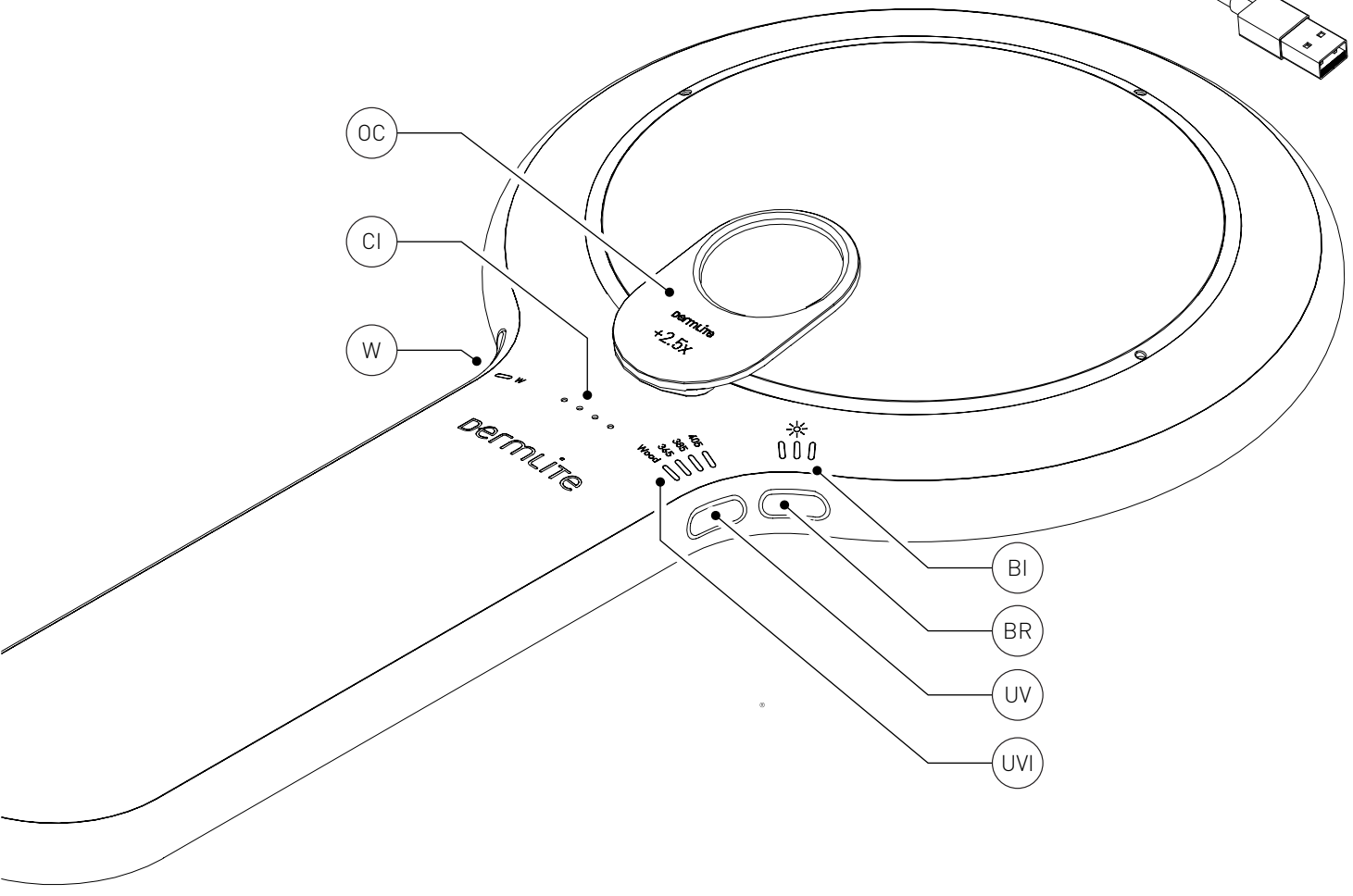

REF DLU2

# Instructions

## Intended Use

This device is intended for medical purposes to illuminate body surfaces. It is used for non-invasive visual examination of intact skin.

This battery-operated product is designed for external examination only within professional healthcare facilities by medical professionals.

Check the correct operation of the device before use!

Do not use the device if there are visible signs of damage, malfunction, or the device operates in a manner not consistent with what is described in this instructions for use.

**CAUTION:** Do not look directly into the LED light. Patients must close their eyes during examinations.

**CAUTION:** UV Light. Do not look directly at light.

**CAUTION:** This product contains magnets. Do not use in proximity of pacemakers or other magnetically sensitive devices or items.

In case of a serious incident with the use of this device, notify DermLite immediately and, if required by local regulations, your national health authority.

**CAUTION:** Do not use the device in fire or explosive risk area (e.g. oxygen-rich environment).

**WARNING:** This product can expose you to chemicals including methylene chloride and hexavalent chromium, which are known to the State of California to cause cancer or reproductive toxicity. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

### Electromagnetic Compatibility

This device complies with the EMC Emissions and Immunity level requirements of the standard IEC 60601-1-2:2014. The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required), this equipment might not offer adequate protection to radio frequency communication services, and the user might need to take mitigation measures, such as relocating or re-orienting the equipment.

**WARNING:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

**WARNING:** Use of accessories other than those provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

**WARNING:** Portable RF transmitters should be used no closer than 30 cm (12 inches) to any part of the device. Otherwise, degradation of the performance of

this equipment could result.

Lumio 2 is intended for skin examinations with magnification, polarized white light, and a range of UV light wavelengths.

Hold Lumio about 75-150 mm (3-6") above the skin to be examined. Press the left button (W) to power on the device in polarized white light mode. Pressing the left button (W) again – while the device is on – toggles between white light and the most recently-used UV lighting mode. To choose between UV light wavelengths, tap the (UV) button until the UV indicator (UVI) corresponding with the desired wavelength illuminates. Note that the “Wood” mode is a blend of UV wavelengths that approximates the spectrum of a classic Wood’s lamp. Choose from one of three brightness levels by pressing the brightness button (BR). The selected brightness level corresponds with the number of brightness indicators (BI) illuminated.

Look through the lens with your eye placed 100-150mm (4-6”) above the device. Move the device closer or farther away from the skin to obtain the desired image focus and field of view.

To turn off the unit, push and hold any button about 1 second. In order to save power, the device will turn off automatically after 3 minutes.

### OptiClip™ Accessory

Lumio 2 includes two magnetic accessories designed to optically enhance the examination. In order to increase the contrast of fluorescent features during UV examinations, magnetically attach the OptiClip (OC) labeled “495nm LONG PASS” at the dual attachment points (AP). In order to increase the main lens magnification by an additional 2.5x, use the “+2.5x” clip. One OptiClip may be stored in the handle’s recessed OptiClip Storage (OS).

### Battery and Charging

**CAUTION:** This device uses a special 3.7V 4000 mAh lithium ion battery, which can only be purchased from DermLite or from an authorized DermLite dealer. Do not under any circumstances use another battery other than the one designed for this unit.

After hundreds of uses the battery will deteriorate, which may result in reduced capacity and/or lithium-ion cell expansion. Monitor signs of a deteriorated battery which may include reduced battery life, unexpected shutdowns, erratic behavior, or sudden power loss. If observed, stop use of the device and do not charge the battery. Replace the battery and dispose of the original in accordance with local regulations. This device is equipped with a four-level Charge Indicator (CI). When the unit is turned on and fully charged, all four LEDs light up. When less than 75% of battery life remains, three LEDs are illuminated. If

more than half the battery is drained, two LEDs are on, while a single LED indicates that less than 25% of battery capacity remains. To charge your device, connect the supplied USB-C to USB cable to the Charging Port (CP) and any IEC 60950-1 compliant (5V) USB port. The Charge Indicator (CI) pulses to indicate charging, with the number of pulsing LEDs corresponding with charge level. When charging is complete, all four indicator LEDs illuminate. After years of use, you may wish to re-place the battery, which is only available directly from DermLite or an authorized DermLite dealer. To change the battery, contact DermLite or your local DermLite dealer.

### Troubleshooting

Please check [www.dermlite.com](http://www.dermlite.com) for the most current troubleshooting information. If your device requires servicing, visit [www.dermlite.com/service](http://www.dermlite.com/service) or contact your local DermLite dealer.

### Care and Maintenance

**CAUTION:** No modification of this equipment is allowed.

Your device is designed for trouble-free operation. Repairs shall be made only by qualified service personnel.

### Cleaning

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.) prior to use on a patient. The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

**Warranty:** 5 years for parts and labor. The battery is warranted for 1 year.

### Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

### The Lumio 2 set includes:

Lumio 2 scope (polarized white LEDs, 365nm LEDs, 385nm LEDs, 405nm LEDs, 100mm 2.3x primary lens), 2.5x add-on lens, 495nm add-on bandpass filter, protective pouch, USB to USB-C charging cable

### Technical Description

Visit [www.dermlite.com/technical](http://www.dermlite.com/technical) or contact your local DermLite dealer.

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DermLite LLC
E. C. REP Ltd
Responsible Person Limited