
DERMLITE HUD®

English	3
Bosanski	4
Čeština	5
Dansk	6
Deutsch	7
Eestlane	8
Español	9
Français	10
Gaeilge	11
Hrvatski	12
Italiano	13
Latvietis	14
Lietuvis	15
Magyar	16
Nederlands	17
Polskie	18
Português	19
Română	20
Slovenščina	21
Slovenský	22
Suomi	23
Svenska	24
Türk	25
Ελληνικά	26
български	27
русский	28
Український	29
한국어	30
日本語	31
简体中文	32

dermlite[®] HUD[®]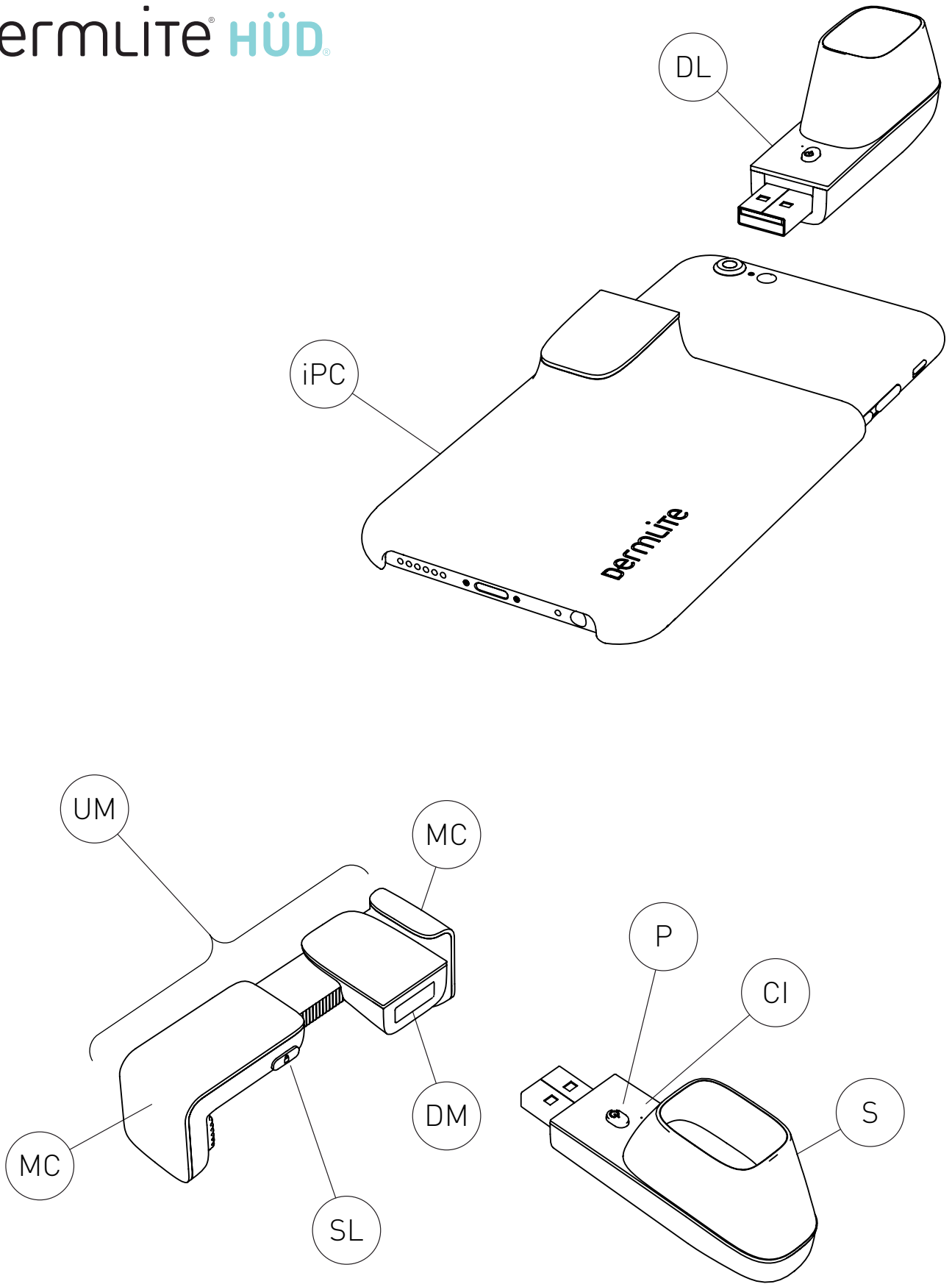

Instructions

Intended Use

This device is intended for medical purposes to illuminate body surfaces. It is used for non-invasive visual examination of intact skin.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

CAUTION: Do not look directly into the LED light. Patients must close their eyes during examinations.

In case of a serious incident with the use of this device, notify DermLite immediately and, if required by local regulations, your national health authority.

CAUTION: Do not use the device in fire or explosive risk area (e.g. oxygen-rich environment).

WARNING: This product can expose you to chemicals including methylene chloride and hexavalent chromium, which are known to the State of California to cause cancer or reproductive toxicity. For more information go to www.P65Warnings.ca.gov.

Electromagnetic Compatibility

This device complies with the EMC Emissions and Immunity level requirements of the standard IEC 60601-1-2:2014. The emission characteristics of this equipment make it suitable for use in professional healthcare environment as well as residential environment (CISPR 11 Class B). This equipment offers adequate protection to radio communication service. In the rare event of interference to the radio communication service, the user might need to take mitigation measures, such as relocating or re-orienting equipment.

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

WARNING: Use of accessories other than those provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING: Portable RF transmitters should be used no closer than 30 cm (12 inches) to any part of the device. Otherwise, degradation of the performance of this equipment could result.

IMPORTANT: Before initial use, please charge the internal lithium ion battery. See "Battery and Charging" for details.

HÜD is a dermatoscope with polarized light designed for smartphone photography of skin lesions.

DermLite HÜD fits on virtually any smartphone 2.3"- 3.1" (58-78mm) in width. For best results, remove your smartphone case. To attach the device, place the Universal Mount (UM) over the back of your smartphone and firmly squeeze the Mount Clamps (MC) together. Move the Slide Lock (SL) into the locked position to hold the Mount Clamps (MC) in place. Slide the DermLite Mount (DM) left or right such that it is horizontally centered over your smartphone

camera. Plug the DermLite dermatoscope into the DermLite Mount (DM). Adjust the vertical position such that the dermatoscope is vertically centered over your phone's camera.

Photography

Once you have chosen a skin lesion to photograph, lightly moisten it and its surrounding skin with hand sanitizer or a similar liquid. This increases the translucency of the skin and improves the quality of the image. With this product attached to your smartphone, place the black spacer (S) directly on the skin and push the power button (P) to turn on the DermLite dermatoscope (DL). Capture the image with your smartphone. Reapply moisture to flatten any hairs as needed. You may have to carefully shave the area.

To conserve energy, this device is designed to automatically turn off after 1 minute. The battery can be expected to supply power for about 30 one-minute sessions. If the device turns off prior to 1 minute, it may need to be recharged. To charge your device, plug it into any IEC 60950-1 compliant (5V) USB charger. While charging, the Charge Indicator (CI) will light up red and, when fully charged, green.

Troubleshooting

Please check www.dermlite.com for the most current troubleshooting information. If your device requires servicing, visit www.dermlite.com/service or contact your local DermLite dealer.

Care and Maintenance

CAUTION: No modification of this equipment is allowed. Your device is designed for trouble-free operation. Repairs shall be made only by qualified service personnel.

Cleaning

The exterior of your device, except the optical parts, may be wiped clean with isopropyl alcohol (70% vol.) prior to use on a patient. The lens should be treated as high-quality photographic equipment and should be cleaned with standard lens cleaning equipment and protected from harmful chemicals. Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

Warranty: 2 years for parts and labor. The battery is warranted for 1 year.

Disposal

This device contains electronics and a lithium-ion battery that must be separated for disposal and may not be disposed of with general household waste. Please observe local disposal regulations.

This set includes a DermLite dermatoscope and a Universal Mount.

Technical Description

Visit www.dermlite.com/technical or contact your local DermLite dealer.

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DermLite LLC 31521 Rancho Viejo Rd Ste 104
E. C. REP Ltd